



**WEB DICOM Viewer MedDream**

**Step By Step**

**Installation Instruction**

Download - Softneta - Windows Internet Explorer

http://www.softneta.com/meddreamdownload.html

Google

Download - Softneta

Home Feeds (1) Print Page Tools

LT

# Softneta

home | sitemap | contacts

| About Company | Products | Downloads | Contacts |

## Products

- PACS Server
- MedDream  
Viewer for PacsOne

- Features
- Screenshots
- FAQ
- Download**

- MedReport  
Medical Notes for PacsOne
- SendToPACS  
dicom converter, dicom sender

## Download

### MedDream WEB DICOM Viewer for PacsOne

NEW user interface in MedDream version 2.01. PacsOne 6.2.1 support.

ver	date	download	size	description
2.01	2009-10-05	<a href="#">MedDream-2.01.zip</a>	8MB	MedDream Windows version package for PacsOne which contains a DEMO license.
	2009-10-05	<a href="#">quick_install-2.01.txt</a>	1KB	Quick Install Manual
	2009-10-05	<a href="#">meddream-2.01.pdf</a>		
	2009-10-05	<a href="#">conformance-2.01.pdf</a>	88KB	DICOM Conformance Statement

Older version:

ver	date	download	size	description
1.13	2009-08-05	<a href="#">MedDream-1.13.zip</a>	7MB	MedDream Windows version package for PacsOne which contains a DEMO license.
	2009-08-05	<a href="#">quick_install-1.13.txt</a>	1KB	Quick Install Manual

Download

http://www.softneta.com/files/meddream/MedDream-2.01.zip

Internet 100%

My Computer

My Network Places

MedDream-2.0  
1.zip

**Extract**

Folder window: MedDream-2.01\meddream

Address: MedDream-2.01\meddream

cdrtools	history.js	search.php
dcmk	history.swf	sharedData.php
ffmpeg	home.php	system.php
flashservices	index.php	study.php
log	login.html	thumbnail.php
temp	login.php	video.php
AC_OETags.js	login.png	xml2array.php
applet.php	logoff.php	
authdb.php	meddream.pdf	
dicom.php	meddream.php	
disconnect.php	meddream.swf	
export.php	php4.3_meddream.dll	
favicon.ico	php4.4_meddream.dll	
favicon.png	php5.0_meddream.dll	
flv.bat	php5.1_meddream.dll	
flv.php	php5.2_meddream.dll	
history.htm	report.php	

41 objects selected

Folder window: C:\Program Files\PacsOne\php\meddream




Address: C:\Program Files\PacsOne\php\meddream

cdrtools	history.js	search.php
dcmk	history.swf	sharedData.php
ffmpeg	home.php	system.php
flashservices	index.php	study.php
log	login.html	thumbnail.php
temp	login.php	video.php
AC_OETags.js	login.png	xml2array.php
applet.php	logoff.php	
authdb.php	meddream.pdf	
dicom.php	meddream.php	
connect.php	meddream.swf	
...nect.php	php4.3_meddream.dll	
...:php	php4.4_meddream.dll	
...avicon.ico	php5.0_meddream.dll	
...avicon.png	php5.1_meddream.dll	
flv.bat	php5.2_meddream.dll	
flv.php	report.php	
history.htm		


41 objects 3.48 MB My Computer

**COPY**



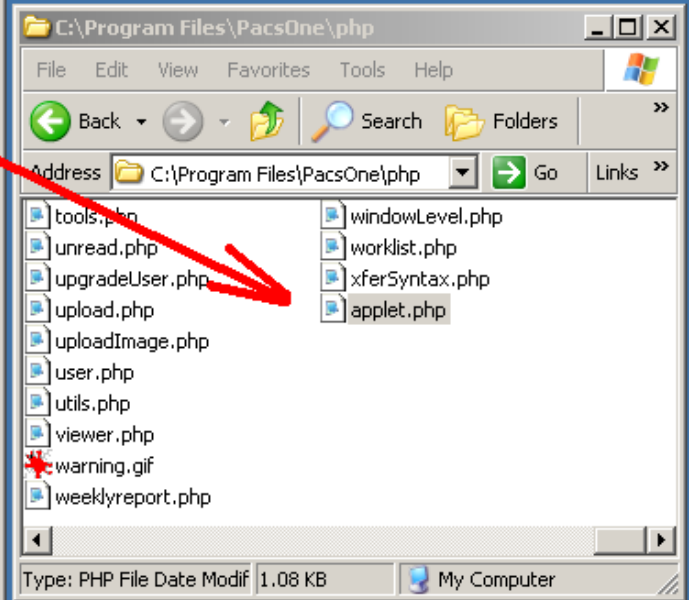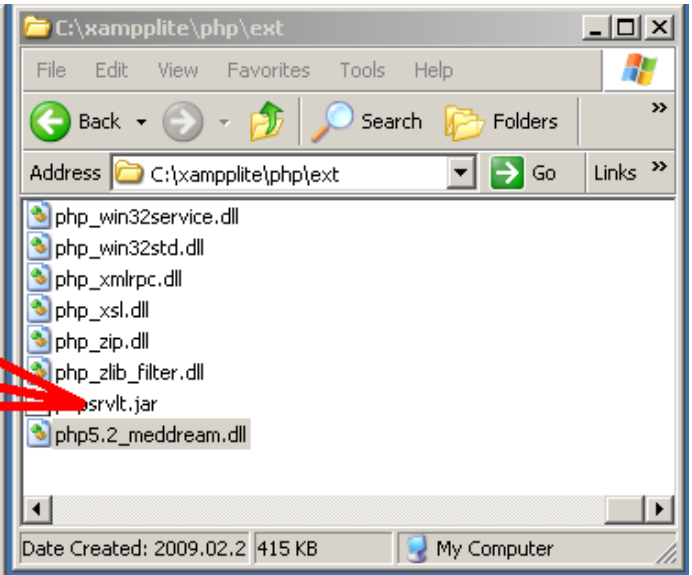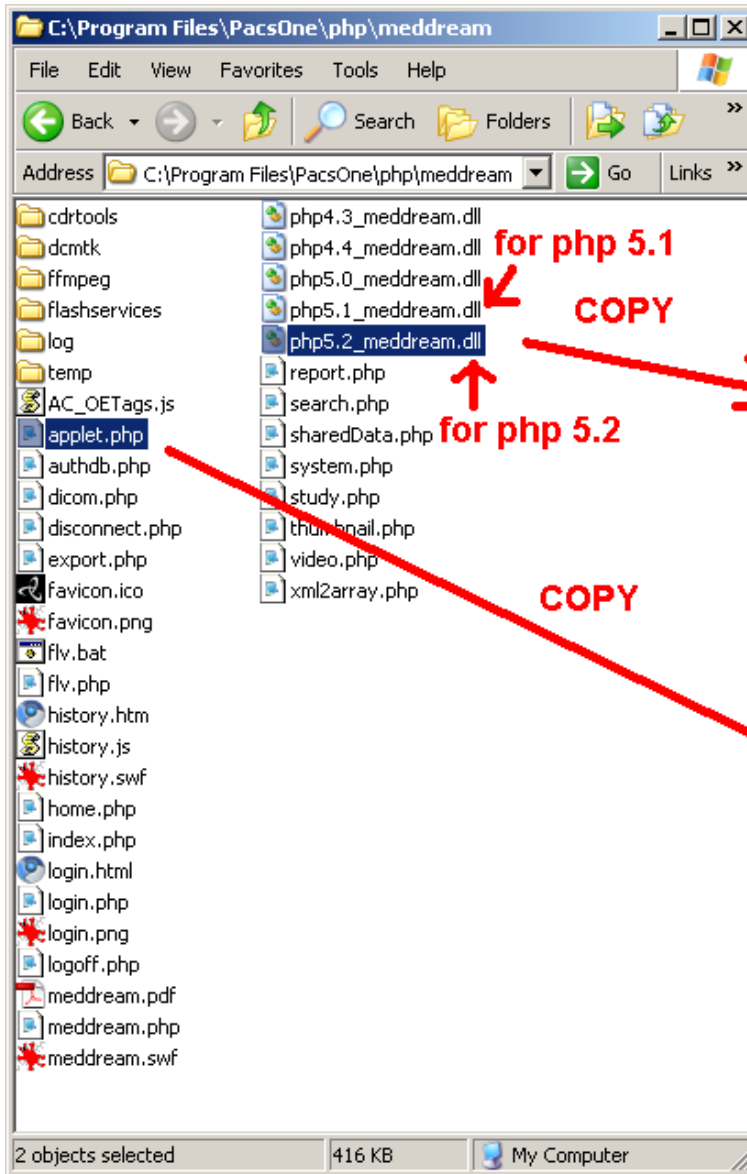



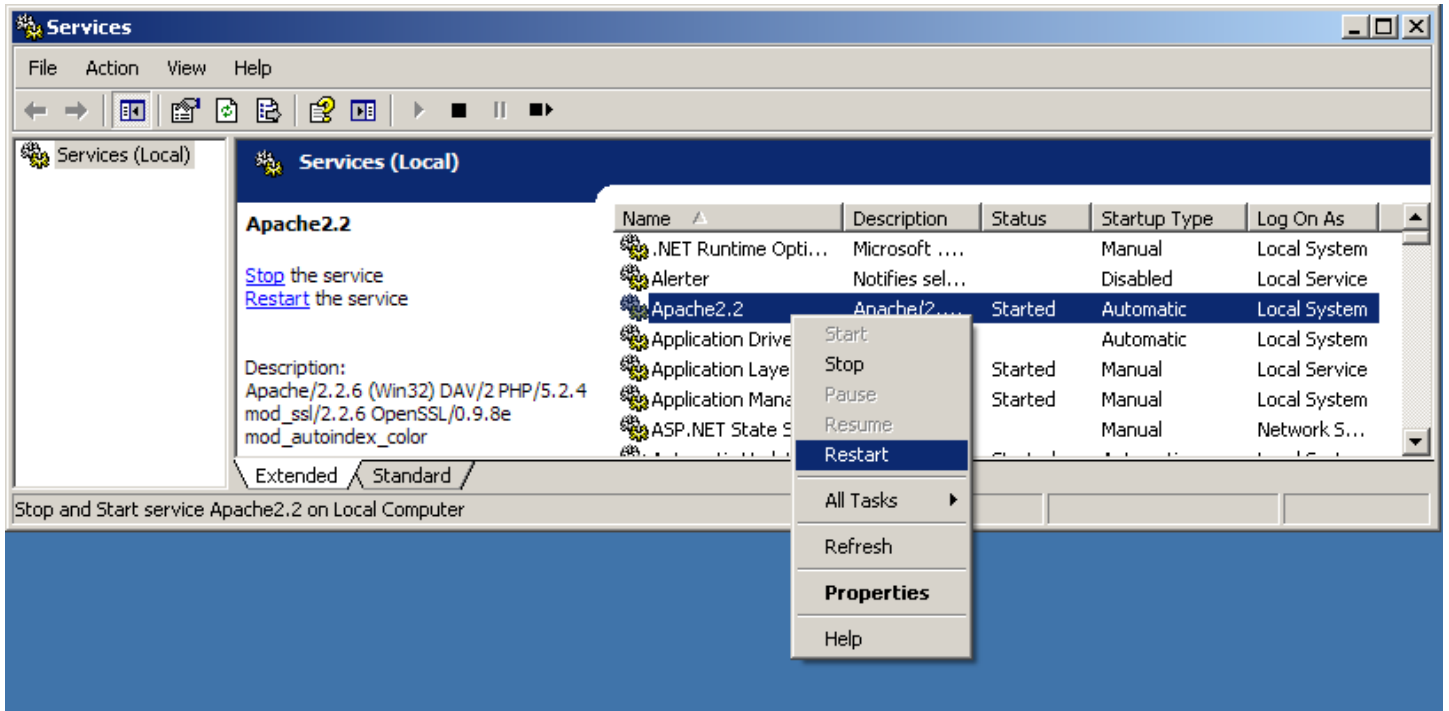

MedDream - Windows Internet Explorer

http://127.0.0.1/pacsone/meddream/home.php

Google

MedDream

Home Feeds (1) Print Page Tools

Copyright 2009 (c) Softneta  
MedDream 2.01

Softneta® **MedDream**  
WEB DICOM Viewer

**Database:**

**Username:**

**Password:**

Login

Done Internet 100%

