

## Quick Install Guide for MedDream DICOM Viewer 7.5.1: Standard DB on PostgreSQL integration

**Important:** please note that instructions above are for Windows only.

### 1. Install PostgreSQL:

<https://get.enterprisedb.com/postgresql/postgresql-10.8-4-windows-x64.exe>

*During the PostgreSQL server configuration remember the port and superuser password;*

### 2. Install MedDream:

<https://www.softneta.com/files/meddreamviewer/751/MDPACK-7.5.1-200410.exe>

### 3. Install MedDream DB:

<https://www.softneta.com/files/meddreamviewer/751/MedDreamViewer-7.5.1-DB-PostgreSQL.zip>

**3.1. Unzip** MedDreamViewer-7.5.1-DB-PostgreSQL.zip to C:\MDPACK\MedDreamViewer-7.5.1-DB-PostgreSQL

**3.2. Install** MedDream DB to MySQL by executing C:\MDPACK\MedDreamViewer-7.5.1-DB-PostgreSQL\PostgreSQL-DB\meddream-db-create.bat

*You will be asked to enter a new password for the user "meddream". Remember it.*

*Immediately after that you will be asked to enter the current password for the superuser.*

**3.3. Insert** test data to MedDream DB by executing C:\MDPACK\MedDreamViewer-7.5.1-DB-PostgreSQL\PostgreSQL-DB\meddream-db-insert.bat

*you will be asked to enter the password for the superuser.*

### 4. Configure MedDream:

**4.1. Copy and replace** c:\MDPACK\MedDreamViewer-7.5.1-DB-PostgreSQL\MDPACK folder to C:\MDPACK

**4.2. Configure** meddream DB by entering user "meddream" password in C:\MDPACK\MedDream\application.properties file:  
com.softneta.meddream.pacs.configurations[0].password={Password} #line 382

### 5. Restart windows services (Run in CMD line: services.msc):

#### 5.1. Restart MedDream

#### You can connect to MedDream with Logins:

http://127.0.0.1/

User name: user

Pass: user

**6. URL Integration:**

<http://127.0.0.1/?study=1.2.826.0.1.3680043.8.1055.1.20160224130121426.315299872.5643427>